

HOW TO CONTROL LIFE-THREATENING BLEEDING WITH CELOX™ GAUZE

CELOX™ GAUZE
MADE TO SAVE LIVES



1. Tear open pack. Before application identify and apply direct pressure on the main part of the bleeding then remove excess blood where practical. Take out the Celox Z-Fold Gauze and take hold of one end with the other hand.



2. Tightly pack the unfolding Celox Z-Fold Gauze directly to the source of bleeding. Pack remaining wound with Celox Z-Fold Gauze or standard gauze above skin level. Excess Celox Z-Fold Gauze can be torn or cut if necessary.



3. Apply FIRM pressure directly to the wound for 3 minutes. If bleeding persists apply additional pressure for an additional 3 minutes.



4. Wrap and tie with a bandage so as to maintain pressure on the wound.

5. Dispose any remaining Celox Z-Fold Gauze according to local standard protocols for biological waste.

6. Transfer patient to medical facilities as soon as possible.

7. Show empty pack to medical personnel.



For full details, warnings and removal instructions refer to manufacturer's instructions for use.

Copyright © 2022 Medtrade Products Ltd. CELOX™ is a trademark of Medtrade Products Ltd. All rights reserved.