



FOR TEMPORARY EXTERNAL USE ONLY



# CELOX™

## Hemostatic Granules

**Stops Bleeding Fast and Works even in the cold\***

TEMPORARY EMERGENCY TREATMENT  
FOR BLEEDING WOUNDS

Indicated for the local management of bleeding such as lacerations, minor cuts and abrasions  
Clots blood treated with blood thinning drugs (Heparin, Warfarin/Coumadin)

\*Laboratory tests have shown the product will clot in blood temperatures down to 56 deg F. Data on file at Medtrade Products Ltd.

**Net Wt 2g (0.07oz)**



Do not re-use



Do not re-sterilize



Caution



Keep dry



Sterilized using irradiation



Do not use if package is damaged

See Instructions for Use on the back

and data

45 x 17 mm  
UDI 2D barcode

 Manufactured by: Medtrade Products Ltd, Electra House, Crewe Business Park, Crewe, CW1 6GL, UK.

MTP-20-1506 Issue Date: 21 Feb 2020

[www.celoxmedical.com](http://www.celoxmedical.com)

**REF** FG08830441

- Warnings:** For temporary external use only. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan from shrimp shell. Allergy studies show no adverse reactions to date. Data on file at Medtrade Products Ltd.
- Removal:** Once bleeding has stopped, granules can be washed from the wound using water or saline.
1. Blot away any excess blood, water or dirt from the wound with sterile gauze.
  2. Tear open CELOX™ pack.
  3. Pour entire contents of pack directly into wound, use additional packs if required.
  4. Quickly cover with sterile gauze or bandage and apply FIRM direct pressure for up to 5 minutes, if bleeding persists apply pressure for 5 more minutes.
  5. Wrap with a bandage. Discard any unused granules after opening.
  6. If in doubt and bleeding continues, seek medical attention as soon as possible. Show empty pack to medical personnel.

### Instructions for Use: