

TEAR

For Emergency Temporary External Use Only

TEAR

CELOX™ GAUZE

Z-Fold Hemostatic Gauze

One Gauze Strip 3in x 5ft (7.6cm x 1.5m)
To Control Moderate-To-Severe Bleeding

Works in Hypothermic Conditions*



Do not
re-use



Do not
resterilize



Sterilized using
irradiation



Caution



Do not use if
package is
damaged

Rx only

Instructions for Use on Back

*Proven in laboratory trials to be effective in blood temperatures down to 56°F (hypothermic).
Data on file at Medtrade Products Ltd.

TEAR

TEAR

TEAR

FOR USE BY TRAINED EMERGENCY RESPONDERS

Instructions for Use:



1. Identify the point of bleeding and apply direct pressure. Swab excess blood where practical.



3. Tightly pack the unfolding CELOX™ Z-Fold Gauze directly to the source of bleeding, feeding in the CELOX™ Gauze until the wound is filled above skin level. Excess CELOX™ Z-Fold Gauze can be cut or torn if necessary.



5. Secure with a dressing to maintain pressure on the wound. Discard any remaining CELOX™ Z-Fold Gauze.



2. Tear open pouch. Take out the CELOX™ Z-Fold Gauze and take hold of one end of Z-folded gauze.



4. Apply FIRM pressure directly to the wound for 3 minutes. If bleeding persists apply additional pressure until bleeding stops.

6. Transfer patient to Medical Facility as soon as possible.

7. Show empty pack to medical personnel.

ATTENTION MEDICAL FACILITY PERSONNEL

1. Physically remove CELOX™ Z-Fold Gauze from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution.
3. Proceed with normal irrigation and/or suction.
4. Ensure all product is removed from the wound prior to initiation of wound treatment.

Warnings: Device is not intended for surgical use. Do not use in abdominal wounds and wounds not amenable to pressure. Loss of sterility potentially poses a risk of infection. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan from shellfish. Allergy studies show no adverse reaction to date. Data on file at Medtrade Products Ltd.

Manufactured by:
Medtrade Products Ltd, Electra House,
Crewe Business Park, Crewe, CW1 6GL, UK.
www.celoxmedical.com
Issue Date: 24 Jan 2020. Artwork Ref: MTP-19-1423

REF FG08838021

TEAR

TEAR