

◀ TEAR

TEAR ▶

For Emergency Temporary External Use Only

CELOX[™] RAPID RIBBON**Hemostatic Z-Fold Gauze**

One Gauze Strip 1 in x 5ft (2.5cm x 1.5m)

Dressing Indicated to Control Moderate-To-Severe Bleeding

Chito-R[™] Technology, Stops Bleeding Fast

Works in Hypothermic Conditions*

Do not
re-useDo not
resterilizeSterilized using
irradiation

Caution

Do not use if
package is
damaged**Rx Only**

Instructions for Use on Back

*Proven in laboratory trials to stop major arterial bleeding by coagulating the blood within 60 seconds, reducing blood loss. Granules shown to be effective in blood temperatures down to 56 degF (hypothermic). Data on file at Medtrade Products Ltd.

◀ TEAR

FOR USE BY TRAINED EMERGENCY RESPONDERS

TEAR ▶

Instructions for Use:

- 1 Identify the point of bleeding and apply direct pressure. Swab excess blood where practical.
- 2 Tear open pack. Take out the Celox Rapid Ribbon and take one end of Z-folded gauze.
- 3 Tightly pack the unfolding Celox Rapid Ribbon directly to the source of the bleeding, feeding in the Celox Rapid Ribbon until the wound is filled above skin level. Excess Celox Rapid Ribbon can be cut or torn if necessary.
- 4 Apply FIRM pressure directly to the wound for 60 seconds. If bleeding persists apply additional pressure until bleeding stops.
- 5 Secure with a dressing to maintain pressure on the wound. Discard any remaining Celox Rapid Ribbon.
- 6 Transfer patient to medical facility as soon as possible.
- 7 Show empty pack to medical personnel.



SCAN ME

Used our product?
Let us know
what you think**ATTENTION MEDICAL FACILITY PERSONNEL.**

- 1 Physically remove Celox Rapid Ribbon from the wound and any loose surface granules.
- 2 Fully flood entire wound area with sterile saline irrigation solution.
- 3 Proceed with normal irrigation and / or suction.
- 4 Ensure all product is removed from the wound prior to initiation of wound treatment.

REF FG08839041

Artwork Ref: MTP-20-1815
Issue Date: 07/Oct/2020

Warnings: Device is not intended for surgical use. Do not use in abdominal wounds and wounds not amenable to pressure. Loss of sterility potentially poses a risk of infection. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan from shellfish. Allergy studies show no adverse reaction to date. Data on file at Medtrade Products Ltd.

Manufactured by: Medtrade Products Ltd, Electra House,
Crewe Business Park, Crewe, CW1 6GL, UK. www.celoxmedical.com
CELOX[™] is a trademark of Medtrade Products Ltd. All rights reserved.

50 x 20 mm
UDI 2D barcode
and data