

TEAR

CELOX[™] RAPID

X-Ray detectable Z-Fold Hemostatic Gauze One Gauze Strip 3in x 5ft (7.6cm x 1.5m)

Indications for use: Celox Rapid X-Ray detectable Z-Fold Hemostatic Gauze is indicated for temporary external use to control moderate to severe bleeding.

May also be indicated for temporary external use to control bleeding of lacerations, minor cuts, and abrasions. Chito-R[™] Technology, Stops Bleeding*

FOR TEMPORARY EXTERNAL USE ONLY

Can be left in place for up to 72 hrs



Caution

STERILE R

Sterilized using irradiation



Do not re-use



Do not re-sterilize

MR

MR Safe

X-Ray
Detectable
Gauze



Do not use if package is damaged



Single sterile barrier system

MD

Medical Device



Instructions for use on back

Rx Only

* Proven in laboratory trials to stop major arterial bleeding by coagulating blood within 60 seconds, reducing blood loss. Granules shown to be effective in blood temperatures down to 56°F. Data on file at Medtrade Products Ltd.

TEAR

TEAR

TEAR

TEAR

FOR USE BY TRAINED EMERGENCY RESPONDERS

Instructions for Use:



1 Identify the point of bleeding and apply direct pressure. Swab excess blood where practical.



2 Tear open pouch. Remove the Celox Rapid and take one end of Z-folded gauze.



3 Tightly pack the unfolding Celox Rapid directly to the source of the bleeding, feeding in the Celox Rapid until the wound is packed above skin level. Excess Celox Rapid can be cut or torn if necessary.



4 Apply FIRM pressure directly to the wound for 60 seconds. If bleeding persists, apply additional pressure until bleeding stops.



5 Secure with a dressing to maintain pressure on the wound. Discard any remaining Celox Rapid.



6 Transfer patient to medical facility as soon as possible.

7 Show empty pack to medical personnel.

ATTENTION MEDICAL FACILITY PERSONNEL:

1. Physically remove Celox Rapid from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution.
3. Proceed with normal irrigation and / or suction.
4. Ensure all product is removed from the wound prior to initiation of wound treatment.
5. Dispose of removed Celox Rapid and residuals according to local standard protocols for biological waste.

Duration of use: The device and residuals should be removed from the wound within 72 hours from application.

Warnings & Precautions: For external use only. Do not eat. If ingested, drink glass of water to avoid discomfort. Loss of sterility potentially poses a risk of infection. Do not re-sterilize. Avoid inhalation. Do not apply over eyes. If eye irritation occurs, flush with water for 5 minutes. Keep away from children. Re-use could result in risk of cross-infection and reduced performance. NOT for internal use and has NOT been tested for that. **Contains Chitosan from shellfish – Allergy studies show no adverse reaction. Data on file at Medtrade Products Ltd.**

Contraindications: Do not use in abdominal wounds and wounds unamenable to pressure. Device not intended for surgical use.



Medtrade Products Ltd,
Electra House, Crewe Business Park,
Crewe, CW1 6GL, UK.

www.celoxmedical.com

REF

FG08839251
Artwork Ref: MTP-23-2675
Issue Date: 08 Feb 2024

CELOX[™] is a trademark of Medtrade Products Ltd. All rights reserved.

TEAR

TEAR