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FOXSEAL™

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Okklusiv forsegling til åbne sår på thorax

BRUGSANVISNING:

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Medtrade Products Ltd, Electra House, Crewe Business Park, Crewe, CW1 6GL, UK. www.celoxmedical.com

MedEnvoy, Prinses Margrietplantsoen 33 – Suite 123, 2595 AM The Hague, The Netherlands

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FOXSEAL™ is a trademark of Medtrade Products Ltd, all rights reserved.

Occlusive Dressing For Open Chest Wounds

INSTRUCTIONS FOR USE:

1. Wipe dirt/fluid from skin around wound.
2. Grip tab and peel to remove one Foxseal from clear liner. This exposes adhesive side.
3. Place dressing centred on the wound, adhesive side down. Preferably do this while patient exhales as air pressure in the pleural cavity should be less than during inhalation.
4. Firmly press dressing to skin around wound to seal.
5. If another wound is present, repeat steps 1-4 with second seal.
6. Use tab to vent and re-seal, if required.

WARNINGS: For use by trained emergency responders only. As with any chest seal, continuously monitor the casualty for tension pneumothorax and if symptoms develop treat as trained, e.g. briefly remove edge of dressing to vent or use needle decompression. Do not place decompression needle through dressing. This occlusive dressing is not intended to be used to improvise a 3-side sealed (flutter valve) dressing and does not contain a valve. For pre-hospital use only. Do not re-use. Reuse may result in reduced adhesion and/or infection.

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