ROZTRHNITE
HÚZZA LE
TEAR

FOXSEAL™ Vented

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Chest Seal Dressing for Open Chest Wounds

INSTRUCTIONS FOR USE:

1. Wipe dirt/fluid from skin around wound.
2. Grip tab and peel to remove clear liner to expose adhesive side.
3. Place dressing adhesive side down, centrally over the wound.
4. Firmly press dressing to skin around wound to seal.
5. Continue to assess casualty and monitor action of chest seal.
6. Use tab to vent and re-seal if required.
7. Use an additional dressing for each additional chest wound.

WARNINGS: The dressing is intended to prevent tension pneumothorax, but as with any chest seal, continuously monitor the casualty for tension pneumothorax and if symptoms develop treat as trained e.g. briefly remove edge of dressing to vent or use needle decompression. Do not place decompression needle through dressing. Do not place over a protruding object as this may worsen injury or damage vents. Do not place the dressing where it will be occluded by body weight, straps, blankets etc. For pre-hospital use only.

2797

STERILE R

Obsahuje jedno krytie. 13,0 x 13,3 cm / Egy db kötszer tartalmaz. 13,0 cm x 13,3 cm /
Contains one dressing. 13.0cm x 13.3cm

Medtrade Products Ltd, Electra House, Crewe Business Park, Crewe, CW1 6GL, UK. www.celoxmedical.com

MedEnvoy, Prinses Margrietplantsoen 33 – Suite 123, 2595 AM The Hague, The Netherlands

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FOXSEAL™ is a trademark of Medtrade Products Ltd, all rights reserved.

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Obells S.A., Bd. Général Wahis 53, 1030 Brussels, Belgium