




## OMNI-STAT Granules (3g) How to Use/Recommendations for Removal

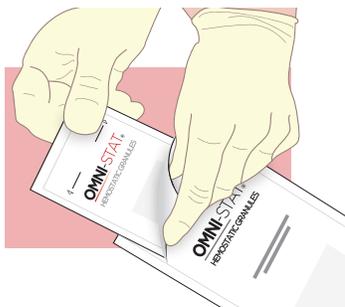


- 1** Peel open Omni-Stat outer pouch and remove inner pouch.

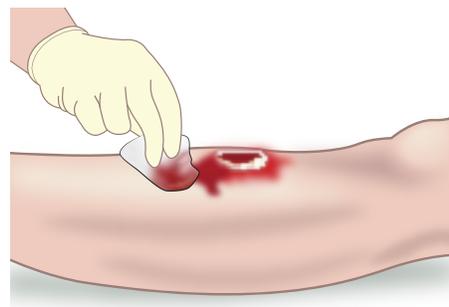


- 2** Tear open Omni-Stat inner pouch. Blot away any excess blood or exudate from the wound.

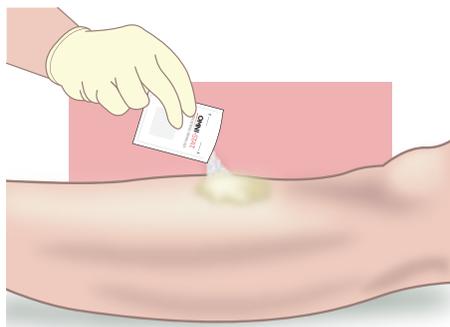


- 3** Pour enough Omni-Stat onto the wound to cover the entire bleeding area. Omni-Stat should be applied directly to the source of bleeding.

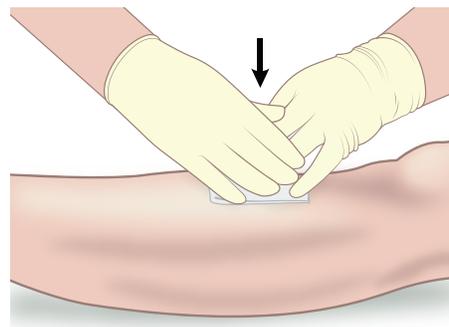

- 4** Quickly cover with gauze and apply firm pressure for 1 to 3 minutes depending on the severity of the bleeding. If bleeding continues apply pressure for 3 more minutes. If further Omni-Stat is required use an additional pouch.

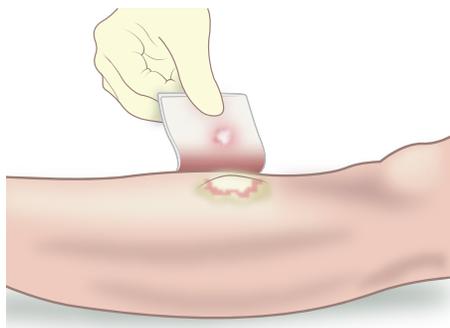

- 5** Five minutes after bleeding has stopped, gently remove any excess Omni-Stat. Use saline if required. Discard any unused Omni-Stat.

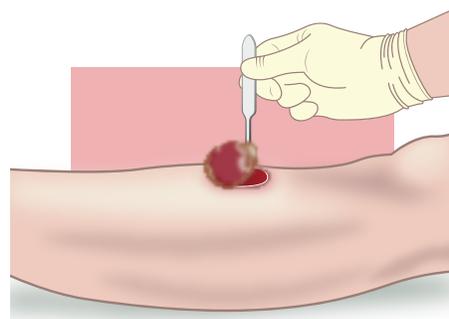

- 6** **Removal Instructions:**  
At first dressing change after use, Omni-Stat should be cleansed and removed from the wound using standard wound cleansing protocols. If required Omni-Stat can be removed (physically) and then any residual irrigated away with water or saline.

**Caution:** For temporary external use only. Do not eat. Do not apply over eyes. Contains chitosan from shellfish, allergy studies show no adverse reaction. Data on file Omni-stat Medical Inc.

## PRODUCT EVALUATION FORM

Evaluation forms can be emailed to [info@omni-stat.com](mailto:info@omni-stat.com) or given to your local sales representative

Facility/Hospital Name: \_\_\_\_\_ Date: \_\_\_\_\_

Provider Name: \_\_\_\_\_ Provider Signature: \_\_\_\_\_

### Product Preparation Used

- |  |  |  |
|--|--|--|
| <input type="checkbox"/> OMNI-STAT Granules (3g)     | <input type="checkbox"/> CELOX™ Z Fold Gauze (5ft) | <input type="checkbox"/> CELOX™ Granules (15g) |
| <input type="checkbox"/> OMNI-STAT Gauze (4in x 4in) | <input type="checkbox"/> CELOX™-A (6g)             | <input type="checkbox"/> CELOX™ RAPID          |


### Type of Wound

- Gunshot
- Laceration/Abrasion
- Puncture
- Blunt Trauma
- Skin Tear
- Diabetic Foot Ulcer
- Pressure Ulcer
- Other: \_\_\_\_\_

### Hold Pressure Time

- 1 minute
- 3 minutes
- 5 minutes
- 10 minutes
- Other: \_\_\_\_\_

### Wound Location

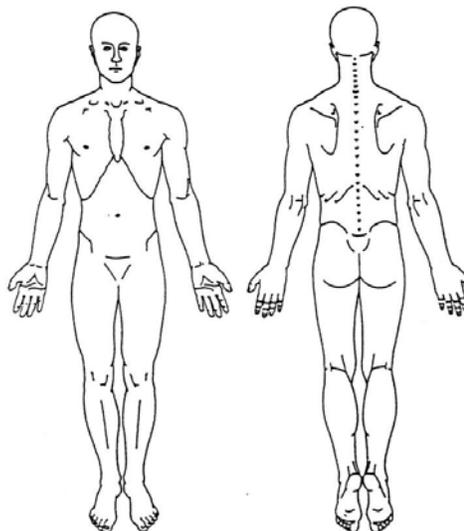

Please circle the location of the wound on the diagram above

### Was the Patient Anticoagulated?

- YES,  
If Yes, please list type  
of anticoagulant: \_\_\_\_\_
- NO

---

---

---

### Was hemostasis achieved?

- YES
- NO

---