

TEAR

FOR TEMPORARY EXTERNAL USE

# SILVAPRO<sup>TM</sup>

## Antibacterial Burn Dressing

### 5in x 9in Dressing



Dressing covers approx. **2%** total body surface area

For the Pre-Hospital Management of 1<sup>st</sup> and 2<sup>nd</sup> Degree Burns with Aquanova Ag Technology

Each dressing contains nominally 0.75% by weight of ionic Silver

INSTRUCTIONS FOR USE ON BACK



Do not re-use



Do not sterilize

**STERILE R**

Sterilized using irradiation



Caution



Do not use if package is damaged



Upper limit of 40°C

Not made with natural rubber latex

**Rx Only**

TEAR

TEAR

## For The Pre-Hospital Management of 1<sup>st</sup> and 2<sup>nd</sup> Degree Burns

### Instructions for Use:

1. Irrigate wound area according to locally established protocols.
2. Tear open dressing pack. Unravel Silvapro Antibacterial Burn Dressing and cover burn. Overlapping at least 2inches onto the surrounding skin.
3. For exuding wounds Silvapro Antibacterial Burn Dressing can be applied dry and gels on contact with wound fluid to form an antibacterial barrier, or alternatively, the dressing should be wetted after application using saline or water to cool the burn.
4. Cover Silvapro Antibacterial Burn Dressing with a sterile loosely woven gauze and secure.

### Once product has been used:

Seek MEDICAL CARE as soon as possible. Show empty pack to medical personnel. Discard unused portion of the Silvapro Antibacterial Burn Dressing after dressing the wound.

### Contraindication:

The dressing is contraindicated for third (3<sup>rd</sup>) degree burns or for use on individuals with known sensitivity to ionic silver.

### Warnings:

Device is not intended for Implantation. For temporary external use only. Do not apply over eyes. Frequent or prolonged use of this product may result in permanent discoloration of skin, due to the ionic silver. Contains chitosan from shellfish. Allergy studies show no adverse reaction to date. Data on file at Medtrade Products Ltd.

In-vitro antibacterial data has shown that Silvapro Antibacterial Burn Dressing inhibits bacterial growth in the dressing. In-vitro Silvapro Antibacterial Burn Dressing has been shown to kill the following organisms, over a 7-day period: *Staphylococcus aureus* (MRSA), *Escherichia coli*, *Pseudomonas aeruginosa*, *Candida albicans*, *Enterococcus faecalis* (VRE), *Staphylococcus aureus*.

50 x 20 mm  
UDI 2D barcode  
and data



Manufactured by: Medtrade Products Ltd,  
Electra House, Crewe Business Park, Crewe, CW1 6GL, UK.  
[www.celoxmedical.com](http://www.celoxmedical.com)

SILVAPRO<sup>TM</sup> is a trademark of Medtrade Products Ltd. All rights reserved.

Issue Date: 09/Nov/2020 Artwork Ref: MTP-20-1831

**REF** FG08825191

TEAR

TEAR