OUVRIR
AUFREISSEN
TEAR

FOXSEAL™ Vented

Chest Seal Dressing for Open Chest Wounds

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en

Chest Seal Dressing for
Open Chest Wounds

INSTRUCTIONS FOR USE:

1. Wipe dirt/fluid from skin around wound.
2. Grip tab and peel to remove clear liner to expose adhesive side.
3. Place dressing adhesive side down, centrally over the wound.
4. Firmly press dressing to skin around wound to seal.
5. Continue to assess casualty and monitor action of chest seal.
6. Use tab to vent and re-seal if required.
7. Use an additional dressing for each additional chest wound.

WARNINGS: For use by trained emergency responders only. This dressing is intended to help prevent tension pneumothorax. As with any chest seal, the casualty must be continuously monitored for sign of tension pneumothorax. If symptoms occur, treat as trained e.g. briefly remove edge of dressing to vent or use needle decompression. Do not place over a protruding object as this may worsen injury or damage vents. Do not place the dressing where it will be occluded by body weight, straps, blankets etc. For pre-hospital use only. Reuse may result in reduced adhesion and/or infection.

Consignes d'utilisation :
voir à l'avant.
Gebrauchsanweisung:
auf der Vorderseite.
Instructions for use:
on front.
[www.colsonmedical.com/
instructions-for-use-ifu](http://www.colsonmedical.com/instructions-for-use-ifu)

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Medtrade Products Ltd, Electra House,
Crewe Business Park, Crewe, CW1 6GL, UK.
www.colsonmedical.com

MedEnvoy Global BV, Prinses
Margrietplantsoen 33 - Suite 123, 2595 AM
The Hague, The Netherlands

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Ref. illustration / Referenz zu den Illustrationen /
Artwork Ref: MTP-26-2964

GTIN-14 barcode
15060206631288

FOXSEAL™ is a trademark of Medtrade Products Ltd. All rights reserved.

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Obelis S.A., Bd. Général Wahis 53,